



The connector has different colours depending on the diameter of the catheter.



All LoFric catheters are coated with the unique Urotonic™ Surface Technology. This ensures that the catheter remains comfortable and safe, both during insertion and withdrawal.

Intended use

The LoFric® Dila-Cath single use hydrophilic dilatation catheter is intended for intermittent dilatation of the urethra.

Sterilization and shelf life

All catheters are sterilized by ethylene oxide gas. Information about the recommended best before date is printed on the package.

Packaging

30 catheters per carton and 4 cartons per shipping box.

Material composition

Hydrophilic surface coating: Polyvinyl pyrrolidone (PVP) and sodium chloride (NaCl).

Catheter: Polyolefin-based elastomer (POBE).

Single package: Lacquered paper, laminated film of Polypropylene (PP) / Polyamide (PA) / Polyethylene (PE).

Product range

Reference	Type	Length	Size
406 16 00	LoFric® Dila-Cath™ Nelaton	40 cm	CH 16
406 18 00	LoFric® Dila-Cath™ Nelaton	40 cm	CH 18

LoFric® Dila-Cath™


Instructions

Important information


Ensure proper training on the dilatation procedure is provided to user.

This is a single use product. Once used, the surface coating will deteriorate and the catheter will no longer be sterile.


Start by washing your hands with soap and water.

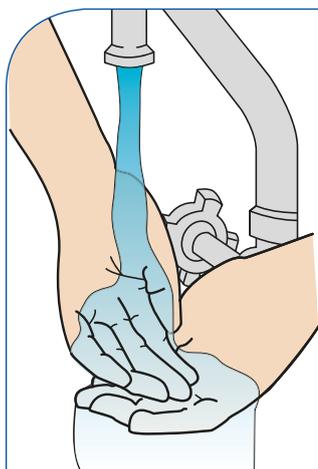


1. Start by washing your hands with soap and water.

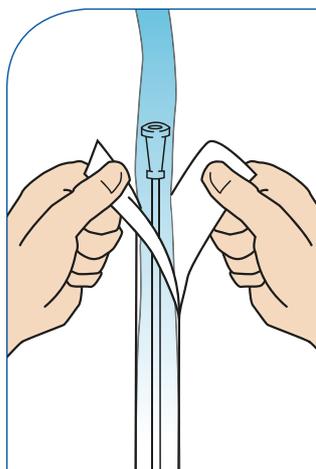


2. Open the pack carefully. Pull the tabs open about 2 inches. Pour water directly into the pack. Use sterile water only as directed by your healthcare provider.

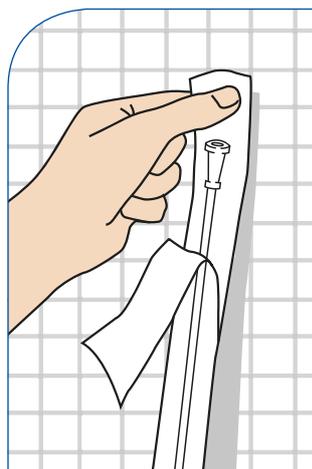


3. The pack has a self-adhesive patch on the back. Remove the blue tab and attach the pack to a sink or other convenient, dry surface.

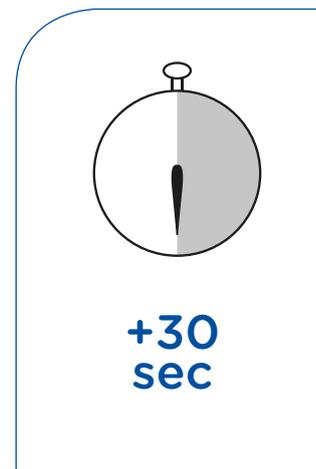

4. Let the catheter soak for at least 30 seconds while you are getting ready.

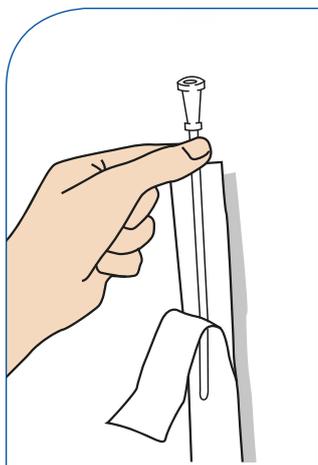

5. The catheter is then ready to be inserted.

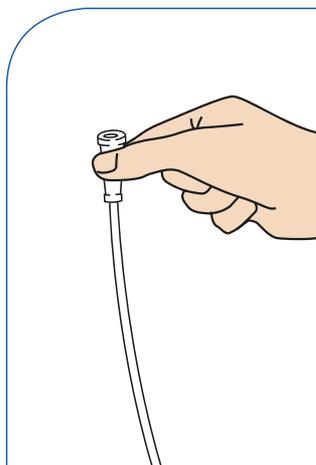

6. Carry out the dilatation procedure as instructed. After use, dispose of the catheter appropriately (local regulations may vary).