

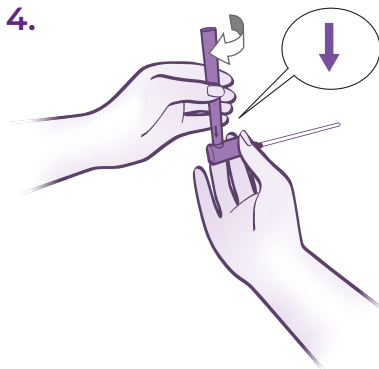


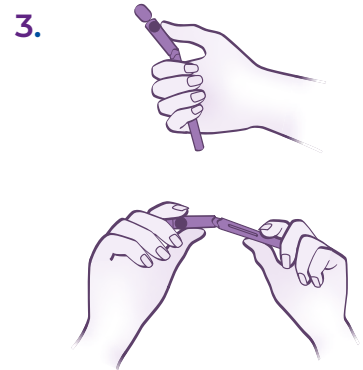

Scan for  
detailed  
instructions

**Wellspect®**  
A REAL DIFFERENCE

# LoFric Elle catheters – quick guide

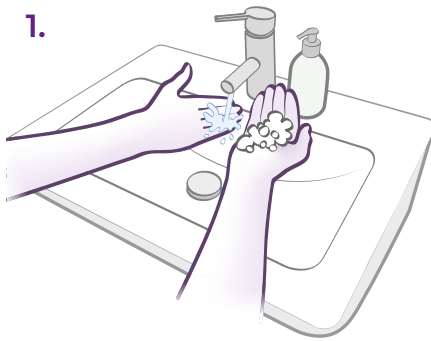

1. Wash your hands thoroughly with soap and water.
2. Open the upper lid.
3. Hold the catheter in your hand and gently bend the upper part to open. Pull the catheter out.
4. Connect the container to the catheter with a gentle twist, make sure the arrow on the container points down.
5. Tilt your pelvis upwards, spread the labia, lift slightly to locate the urethra. The urethra is located just above the vaginal opening.

With the other hand, insert the catheter slowly into your urethral opening, until urine starts to flow, insert slightly more to ensure both eyelets are inside the bladder.

When the urine flow slows to a drip, withdraw the catheter slowly. If urine starts to flow again, stop the withdrawal process and wait until the urine flow stops, to ensure complete bladder emptying.

6. Put the catheter back inside the container. Dispose, or carry it in your bag until disposal. Please note! The container is recyclable.

## Important information

---

LoFric® Elle™ single use, hydrophilic urinary catheter, ready to use.

For female users; adults, adolescents and children.

For short and long term bladder management with intermittent urinary catheterization.

LoFric catheters are for prescription use only.

Ensure to follow the advice and training on the catheterization procedure given by your Healthcare Professional, before use.

Please refer to instructions for use included in the package, for important information and complete instructions.

Contact your prescriber if you experience difficulties.

At Wellspect, we develop innovative products and solutions that make a real difference in the lives of people managing continence care needs, empowering them to live each day to the fullest. We have been a trusted leader in healthcare for over 40 years with our product brands LoFric® and Navina™, and we continue to expand access to our user-friendly solutions. Every day, we strive to maintain the highest quality standards while minimizing the environmental impact of our products. By working closely with users and healthcare professionals, we aim to improve clinical outcomes and enhance quality of life for people around the world.

**Wellspect. A Real Difference.**

For more information about our products and services, please visit [Wellspect.com](https://wellspect.com).

Join the conversation on LinkedIn, X, Facebook and Instagram.

**[wellspect.com](https://wellspect.com)**

Wellspect AB, Aminogatan 1, SE-431 21 Mölndal, Sweden. Phone: +46 (0)31 376 40 00.

LoFric



Manufacturer: Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.  
Phone: +46 31 376 40 00. [www.wellspect.com](https://www.wellspect.com)

